



Established 1972
 By and For the People of Love County
 Richard Barker,
 Administrator

Ambulance Emergency
911


Clinic Appointments
276-2400

Hospital Information
276-3347

Community Resources
211

Domestic Violence Help
276-2042
Hotline - 226-6424

THERAPY DEVICES FOR DRESSING

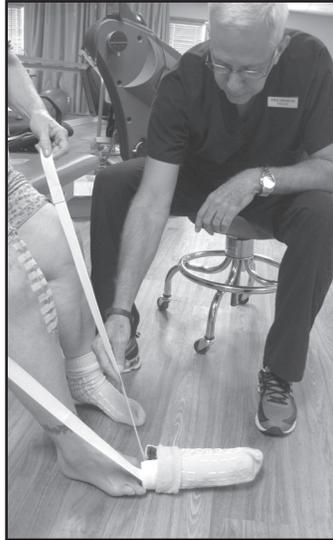

Dressing Without Bending or Stooping: Patient Teresa Strickland, with Occupational Therapist Assistant Bill Graham, demonstrates the sock aid (l) and the reacher/grabber (c) and (r) for adding or removing clothing from an operated leg.

Sock Aid: The sock aid is a tubular device with a nylon interior and a terry cloth exterior. While seated, pull your sock over the device. Place the device on the floor. Slide your foot into the device. Pull up on the side straps. The sock aid comes free after the sock clears the ankle. Lower the device and step away.

Reacher/Grabber: This aluminum device is 26" to 32" long, with a trigger-style handgrip and a serrated jaw. Pull the trigger and the jaws tighten to provide a secure grip for lifting. A small hook mounted to the end aids in retrieving things like clothes or shoes.

We're YOUR Hospital, Clinic and EMS

Mercy Health/Love County

300 Wanda - Marietta, OK 73448

www.mercyhealthlovecounty.com

*The Small-Town Hospital
 with the Big Heart*

